

MAX WOCHER & SON, CINCINNATI, O.

ARTIFICIAL EYES.

In ordering an Artificial Eye the following questions should be answered:

1—Right or left eye? 2—Is the eyeball shrunken or removed? 3—Has an artificial eye been worn? 4—Is the sound eye prominent, very full or sunken? 5—Is the white of the eye clear or of a dark or yellowish tinge? 6—Color of iris. 7—Diameter of iris. 8—Diameter of pupil. 9—Distance from inner to outer canthus. 10—Greatest width of eye

In cases where an artificial eye has been worn, it is desirable, in ordering another, to send the old one, whether broken or intact, as a sample, with answers to the following questions:

11—Is the eye a perfect and comfortable fit? 12—When the eyelids are closed, do they cover it completely? 13—If the eye occasions uneasiness, where is the pain felt? 14—Can this uneasiness be compared to a pressure or to the contact of some irritating substance? 15—Mention if any variations from sample are required in the color or diameter of the iris and pupil.

Directions for Inserting and Removing an Artificial Eye.

To insert the eye, hold it between the forefinger and thumb; wet it by dipping in water. Push its broad outer end under the upper eyelid and slide it up towards its destined position as far as it will readily go; retain it there with the forefinger, and with the finger of the other hand draw down the lower eyelid till the lower edge slips in. The insertion will be facilitated by looking downward.

To remove the eye, depress the lower eyelid with the finger, pass the edge of the thumb-nail, the head of a pin, a little hook or any small blunt instrument under the edge of the eye, lift it forward and let it slip out. Both in inserting and extracting an artificial eye it is best to hold the face over a bed, a soft cushion or the like, so that if the eye should slip out of the fingers it will not be broken in falling.

The artificial eye should be taken out on going to bed and placed in water to cleanse it of the mucus which may be adherent to it. The depression at the top of an artificial eye is for the purpose of preventing the eye from interfering with the action of the superior oblique in cases where this muscle still retains its insertion into the sclerotic

PRICE, Artificial Eyes, taken from stock, each..... \$ 3 00
 " " " made to order, from samples or drawings, each..... \$6 00 to 10 00

Dr. Snellen's "Reform" Artificial Eye, with concavity closed by glass disc giving a broader surface for resting upon the stump of the enucleated eye, assuring greater mobility, prevention of the accumulation of mucus and fluids, and avoidance of the irritation often produced by an eye of the usual make when worn for some time.

PRICE, eachnet \$ 5 00

An extra charge in addition to above prices is made when we fit the eye, depending on amount of time and attention required in fitting.

ALL INSTRUMENTS BEARING THE K.-S. CO.'S



TRADE MARK ARE SOLD UNDER GUARANTEE.

OPHTHALMIC INSTRUMENTS.